

What's New for Users

- Ability to utilize the integration with the Thornhill MOVES® SLC™ when this has been enabled for the user's institution.
 - When enabled for the institution, eNcounter users will now have the ability to turn the Thornhill MOVES SLC functionality on or off via "Device Settings".
 - Ability to launch this functionality from the eNcounter workspace.
 - Ability to view data being generated by the MOVES SLC within the eNcounter workspace (both in the eNcounter and eView user's side).
 - Ability to view data in "Data view" or "Dashboard view" and toggle between the two as needed by the user.
 - Ability to set timeframes for reporting.
 - Ability to view PDF reports including the data viewed during the consult.
 - Ability to push these PDF reports to a specified repository, such as an EHR.
- Ability to utilize SSO via MS Entra ID to sign into eNcounter, when this has been enabled and set up for the institution.
- Addition of option to search by "DOB" Patients Date of Birth for DICOM users.
 - Including the ability for the eNcounter user to turn this search field on or off within "Preferences" under Settings.
- Ability to utilize an "eyeball" icon within the enter password field to be able to see the values entered and confirm accuracy, should the user choose to utilize it. Available in eNcounter, eView and Admin Tool
- Bug fixes to eMDR upload functionality.

What's New for ClearSteth

- No changes for this release.

What's New for Administrators

- **Thornhill MOVES® SLC™ Integration:** Ability to turn the Thornhill MOVES SLC integration functionality on or off for the institution at the GlobalMed Admin level based on the customers need to utilize this functionality, or not.
 - Default to being off/disabled.

- **SSO:** Ability to turn the SSO functionality on or off for the institution at the GlobalMed Admin level based on the customers need to utilize this functionality or not.
 - Includes ability to set up via MS Entra ID.
 - Add and remove users from your institution.
 - Default to off/disabled.